





QuickStart Guide
Wiselion WBS-T007



Quick Start Guide

Introduction



The device needs to be adjusted for maximum accuracy before first use. Please follow the instructions provided here.

- Measure your temperature with another direct contact thermometer you trust.
- Then measure your temperature with the Wiselion digital thermometer.

If for example the temperature of the digital thermometer has a deviation of $+0.5\text{ }^{\circ}\text{C}$ from your thermometer then set $-0.5\text{ }^{\circ}\text{C}$ (**step 4**).



Quick Start Guide

Step 1



Press here to activate the device.



Quick Start Guide

Step 2



Press the "Set button" for about 3 seconds. After you release the button, the screen will be as showed.



Quick Start Guide

Step 3



Press the "Set" button two (2) times.
The screen will show the below indication. (F3)



Quick Start Guide

Step 4

Press the "+" & "-" in order to adjust the setting to the desired value according to the deviation.



Quick Start Guide

Step 5



Finally press the "Set button" two (2) times to complete the set up.





Thank you